EN 3	Irrigation System Instructions for Use
CS 4	Irigační systém Návod k použití
DA 5	Skyllesystem Brugsanvisning
DE 6	Irrigationssystem Gebrauchsanweisung
EL 7	Σύστημα καταιονισμού Οδηγίες χρήσης
ES 8	Sistema de irrigación Instrucciones de uso
FR 9	Système d'irrigation Mode d'emploi
HU 10	Irigációs rendszer Használati utasítás
IT 11	Sistema di irrigazione Istruzioni per l'uso
NL 12	Irrigatiesysteem Gebruiksaanwijzing
NO 13	Irrigasjonssystem Bruksanvisning
PL 14	System do irygacji Instrukcja użycia
PT 15	Sistema de irrigação Instruções de utilização
SV 16	Irrigationssystem Bruksanvisning

T _ U I S _ R E V 1

IRRIGATION SYSTEM

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

Irrigation system with hand pump

INTENDED USE

Used to provide controlled manual irrigation of the ureter, bladder, kidney, or uterine cavity.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

INSTRUCTIONS FOR USE

1. Remove protective covers from spikes.
2. Close blue ON/OFF fittings.
3. Close white regulating clamp at distal end of system.
4. Insert spikes into sterile saline or other irrigation system.
5. Release blue ON/OFF fitting(s).
6. Squeeze and release drip chamber until 1/2 full.
7. Open white regulating clamp.
8. Prime irrigation system.
9. Connect MLLA to endoscope.
10. Use hand pump above white regulating clamp to control irrigation.

NOTE: Before irrigant bag is empty, open blue ON/OFF fitting on opposite solution source and close blue ON/OFF fitting below empty irrigant bag.

NOTE: Viscous solutions, such as Hyskon®, work best when diluted with 70% saline.

NOTE: One “squeeze” of the hand pump instills approximately 25 ml of medium at an average pressure of 6 psi.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.