- EN**
3 **Universa® Loop Drainage Catheter Set**
Instructions for Use
- CS**
4 **Souprava smyčkového katetru Universa®**
Návod k použití
- DA**
5 **Universa® løkkekatetersæt til drænage**
Brugsanvisning
- DE**
6 **Universa® Schlaufen-Drainagekatheter-Set**
Gebrauchsanweisung
- EL**
8 **Σετ καθετήρα με βρόχο παροχέτευσης
Universa®**
Οδηγίες χρήσης
- ES**
9 **Equipo de catéter de drenaje de lazo Universa®**
Instrucciones de uso
- FR**
10 **Set de sonde de drainage à boucle Universa®**
Mode d'emploi
- HU**
12 **Universa® hurokdrenázskatéter-készlet**
Használati utasítás
- IT**
13 **Set con catetere di drenaggio ad
anello Universa®**
Istruzioni per l'uso
- NL**
14 **Universa® lus-drainagekatheterset**
Gebruiksaanwijzing
- NO**
16 **Universa® slyngedrenasjekatetersett**
Bruksanvisning
- PL**
17 **Zestaw cewnika do drenażu z pętlą Universa®**
Instrukcja użycia
- PT**
18 **Conjunto de cateter de drenagem com
ansa Universa®**
Instruções de utilização
- SV**
20 **Universa®-set för dränagekateter med ögla**
Bruksanvisning

UNIVERSA® LOOP DRAINAGE CATHETER SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

The Universa Loop Drainage Catheter is available in two set configurations:

The Introductory Set Includes:

- Loop catheter
- Trocar needle with obturator
- Wire guide
- Dilators
- Straightening stylets
- Connecting tube with one-way stopcock
- Silicone retention disc with plastic tie

The Exchange Set Includes:

- Loop catheter
- Straightening stylets
- Connecting tube with one-way stopcock

INTENDED USE

The Universa Loop Drainage Catheter Set is intended to provide percutaneous urine drainage from the genitourinary system.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- Indwell time should not exceed four weeks. Periodic evaluation of the catheter is advised.
- This device is intended for use by physicians trained and experienced in percutaneous access and placement of drainage catheters. Standard techniques should be employed.
- Do not forcefully advance any component during removal or replacement of the catheter. Carefully remove the components if any resistance is encountered.

INSTRUCTIONS FOR USE

Introductory Set

1. Identify and anesthetize the skin site overlying the collecting system/bladder.
2. Under fluoroscopy or ultrasound, use standard percutaneous technique to place a wire guide into the collecting system/bladder. Confirm placement via free flow of urine.
3. To facilitate passage of the catheter, dilate the musculofascial tract using dilators in sequence from smallest to largest.
4. Advance the straightening stylet into the catheter and secure with the Luer lock.
5. Place the stylet/catheter assembly, pigtail end first, over the external end of the wire guide and advance it into the collecting system/bladder.
6. Confirm catheter position via fluoroscopy or ultrasound.
7. Unlock the stylet, gently twist, and remove the stylet from the catheter.
8. Hold the catheter securely in position and remove the wire guide.
9. Confirm catheter position via fluoroscopy or ultrasound.
10. Use the retention disc to secure the catheter to the skin; secure the catheter to the disc with the plastic tie.
11. Apply tape, suture and dressing as needed.
12. Attach the connecting tube to the catheter and drainage bag.
13. Retain the individually packaged, flexible stylet for catheter removal.

Exchange Set

1. Establish a nephrostomy/suprapubic tract in standard fashion, leaving a wire guide in the collecting system/bladder.
2. Advance the straightening stylet into the catheter and secure with the Luer lock.
3. Place the stylet/catheter assembly over the external end of the wire guide and advance it into the collecting system/bladder.
4. Confirm catheter position via fluoroscopy or ultrasound.
5. Unlock the stylet, gently twist, and remove the stylet from the catheter.
6. Hold the catheter securely in position and remove the wire guide.
7. Confirm catheter position via fluoroscopy or ultrasound.
8. Secure the catheter with tape, suture and dressing as needed.
9. Attach the connecting tube to the catheter and drainage bag.
10. Retain the individually packaged, flexible stylet for catheter removal.

Catheter Removal

1. Place the straightening stylet into the catheter and secure with the Luer lock.
2. If desired, a wire guide may be passed into the collecting system/bladder to maintain access, after the stylet is engaged and before catheter removal.
3. Remove the catheter.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd