- EN**
3 **Universa® Malecot Drainage Catheter Set**
Instructions for Use
- CS**
4 **Souprava katetru Universa® Malecot**
Návod k použití
- DA**
6 **Universa® Malecot drænagekatetersæt**
Brugsanvisning
- DE**
7 **Universa® Malecot-Drainagekatheter-Set**
Gebrauchsanweisung
- EL**
9 **Σετ καθετήρα παροχέτευσης Universa® Malecot**
Οδηγίες χρήσης
- ES**
11 **Equipo de catéter de drenaje Malecot Universa®**
Instrucciones de uso
- FR**
13 **Set de sonde de drainage de Malecot Universa®**
Mode d'emploi
- HU**
15 **Universa® Malecot drenázskatéter-készlet**
Használati utasítás
- IT**
17 **Set con catetere di drenaggio Malecot Universa®**
Istruzioni per l'uso
- NL**
18 **Universa® Malecot drainagekatheterset**
Gebruiksaanwijzing
- NO**
20 **Universa® Malecot drenasjekatetersett**
Bruksanvisning
- PL**
22 **Zestaw cewnika do drenażu Malecot Universa®**
Instrukcja użycia
- PT**
24 **Conjunto de cateter de drenagem Universa® Malecot**
Instruções de utilização
- SV**
25 **Universa®-set för Malecot-dränagekateter**
Bruksanvisning

UNIVERSA® MALECOT DRAINAGE CATHETER SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

The Universa Malecot Drainage Catheter Set is available in three set configurations:

The Introductory Set includes:

- Malecot catheter
- Trocar needle with obturator
- Wire guide
- Dilators
- Straightening stylets
- Connecting tube with one-way stopcock
- Silicone retention disc with plastic tie

The Exchange Set includes:

- Malecot catheter
- Straightening stylets
- Connecting tube with one-way stopcock

The Suprapubic Set includes:

- Malecot catheter
- Hollow needle
- Connecting tube with one-way stopcock

INTENDED USE

The Universa Malecot Drainage Catheter Set is intended to provide percutaneous urine drainage from the genitourinary system.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- Indwell time should not exceed four weeks. Periodic evaluation of the catheter is advised.
- This device is intended for use by physicians trained and experienced in percutaneous access and placement of drainage catheters. Standard techniques should be employed.
- Do not forcefully advance any component during removal or replacement of the catheter. Carefully remove the components if any resistance is encountered.

INSTRUCTIONS FOR USE

Introductory Set

1. Identify and anesthetize skin site overlying the collecting system/bladder.
2. Under fluoroscopy or ultrasound, use standard percutaneous technique to place a wire guide into the collecting system/bladder. Confirm placement via free flow of urine.
3. To facilitate passage of the catheter, dilate the musculofascial tract using dilators in sequence from smallest to largest.
4. Advance the straightening stylet into catheter and secure with Luer lock to close the Malecot wings.
5. Place the stylet/catheter assembly over the external end of wire guide and advance it into the collecting system/bladder.
6. Confirm catheter position via fluoroscopy or ultrasound.
7. Unlock the stylet, gently twist and remove the stylet from the catheter to open the Malecot wings.
8. Hold the catheter securely in position and remove the wire guide.
9. Confirm catheter position via fluoroscopy or ultrasound.
10. Use the retention disc to secure the catheter to the skin; secure the catheter to the disc with the plastic tie.
11. Apply tape, suture, and dressing as needed.
12. Retain the individually packaged, flexible stylet for catheter removal.

Exchange Set

1. Establish a nephrostomy/suprapubic tract in standard fashion, leaving a wire guide in the collecting system/bladder.
2. Advance the straightening stylet into the catheter and secure with the Luer lock to close the Malecot wings.
3. Place the stylet/catheter assembly over the external end of the wire guide and advance it into the collecting system/bladder.
4. Confirm catheter position via fluoroscopy or ultrasound.

5. Unlock the stylet, gently twist, and remove stylet from catheter to unlock the Malecot wings.
6. Hold the catheter securely in position and remove the wire guide.
7. Confirm catheter position via fluoroscopy or ultrasound.
8. Secure the catheter with tape, suture, and dressing as needed.
9. Attach the connecting tube to the catheter and drainage bag.
10. Retain the individually packaged, flexible stylet for catheter removal.

Suprapubic Set

1. Anesthetize the skin at the puncture site.
2. Advance the needle obturator into the catheter tip and secure with the Luer lock to close the Malecot wings.
3. If the bladder is not already distended by urine, fill the bladder with sterile saline.
4. Make a 2 mm incision at the puncture site, approximately 2.5 - 5.0 cm above the symphysis pubis (below the palpable bladder dome).
5. For patients who have not had previous pelvic surgery, insert the catheter vertically into the bladder. For patients who have had previous pelvic surgery, insert the catheter at a 30° angle toward the symphysis pubis.
6. Observe for urine flow. If urine fails to flow spontaneously through the needle obturator, aspirate urine to ensure the catheter is in the bladder.
7. Make sure the catheter is stabilized, then release the Luer lock and rotate the needle obturator counterclockwise to open the Malecot wings.
8. Remove the needle, while being careful not to displace the catheter.
NOTE: Do not re-insert the needle once it has been removed.
9. Attach the connecting tube to the catheter and drainage bag, and allow the bladder to empty.
10. Withdraw the catheter until the Malecot wings meet resistance from the bladder dome.
11. Advance the catheter approximately 2 cm back into the bladder to accommodate patient movement.
12. Secure the catheter with tape, suture and dressing as needed.

Catheter Removal

1. Place the straightening stylet into catheter and secure with the Luer lock to straighten the Malecot wings.
2. If desired, a wire guide may be passed into the collecting system/bladder to maintain access, after the stylet is engaged and before catheter removal.
3. Remove the catheter.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd