UPJ Occlusion Balloon Catheter

Balónkový katetr pro okluzi UPJ

**UPJ (ureteropelvic junction, ureteropelvin overgang)
okklusjonsballonkateter**

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UPJ OCCLUSION BALLOON CATHETER

Used to temporarily occlude the ureteropelvic junction to prevent stone fragments from entering the ureter during percutaneous lithotripsy, and for injection of contrast medium. The removable inflation/injection adapter allows the cystoscope to be removed after the catheter is in place. Intended for one-time use.

DESCRIPTION

- Radiopaque balloon catheter
- Removable inflation/injection adapter
- Syringe

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING UPJ OCCLUSION BALLOON CATHETER

CAUTION: This product contains natural rubber latex which may cause allergic reactions in some individuals.

CAUTION: Using excessive pressure to inflate the balloon on this device can cause the balloon to rupture. Inflation of the balloon 2 or 3 times with an appropriate inflation medium prior to use will allow easier inflation of the balloon.

1. Under x-ray or fluoroscopic control, cystoscopically pass the UPJ Occlusion Balloon Catheter into the ureter and up to the UPJ. **CAUTION: Only water soluble lubricants are recommended for use with the occlusion balloon catheter. Use of other than water soluble lubricants may cause the inflated balloon to burst.**

WARNING: Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.

2. After positioning and before inflating the balloon, remove the cystoscope.
3. Fill the syringe with dilute (25% solution) contrast medium. Eliminate air from the syringe.

CAUTION: Do not overinflate the balloon.

4. Press the syringe tip into the inflation check valve. Using light to moderate thumb pressure, inject the appropriate amount of contrast medium.

CAUTION: Refer to product label or the inflation check valve on the balloon device for appropriate balloon volume.

CAUTION: To prevent the balloon from rupturing, avoid excessive pressure. The amount of inflation may be determined fluoroscopically.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.