Urodynamics Catheters

Urodynamické katetry

Urodynamikkatetre

Urodynamik-Katheter

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T _ U R O D _ R E V 2

URODYNAMICS CATHETERS

Used for cystometry and urethral pressure profilometry. Intended for one-time use.

Catheter lumens are identified as follows:

"fill"	- bladder fill
"B"	- bladder/cystometric pressure
"U"	- urethral pressure
2 ml	- balloon fill

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

Do not use catheter if there is damage to the urethra or bladder, or in the presence of cystitis or bacterial infection.

SUGGESTED INSTRUCTIONS FOR USING URODYNAMICS CATHETER

1. Catheterize the patient in the usual manner.
2. Connect the urodynamic measuring equipment to the appropriate lumen. Proceed with the urodynamic study.
3. Upon completion of the study, dispose of the catheter.

SUGGESTED INSTRUCTIONS FOR USING URODYNAMICS BALLOON CATHETER

1. Catheterize the patient in the usual manner.
WARNING: Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.
2. Fill the syringe with sterile water or saline solution. Eliminate air from the syringe.
CAUTION: Do not overinflate the balloon.
3. To inflate the balloon, instill 2 ml of sterile water or saline solution into the appropriate lumen.
4. Connect the urodynamic measuring equipment to the appropriate lumen. Proceed with the urodynamic study. **NOTE:** Be certain there are no air voids in the line.
5. Deflate the balloon before removing the catheter from the bladder.
6. Upon completion of the study, dispose of the catheter.

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© COOK 2022
Bloomington, IN 47404 U.S.A.