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T _ U U F D _ R E V 1

EN
3
CS
3
DA
4
DE
5
EL
5
ES
6
FR
7
HU
8
IT
9
NL
9
NO
10
PL
11
PT
12
SV
12
ZH
13

INSTRUCTIONS FOR USE OF AQ® HYDROPHILIC DILATORS

NOTE: Prior to use, immerse AQ® dilator in sterile water or isotonic saline to allow the hydrophilic surface to absorb water and become lubricious. This will ease placement under standard conditions.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

URETHRAL DILATORS

1. Introduce a flexible tip of a wire guide into the urethral meatus, and gently manipulate it beyond the obstruction and into the bladder.

NOTE: The wire guide can also be placed through a cystoscope and maintained in position as the scope is removed.

2. Progressing from the smallest to the largest appropriate size, pass the dilators over the wire guide while maintaining the wire guide's position.

URETERAL DILATORS

1. Under x-ray or fluoroscopic control, cystoscopically pass the wire guide the desired distance into the ureter.
2. Progressing from the smallest to the largest appropriate size, pass the dilators over the wire guide while maintaining the wire guide's position. **NOTE:** Dilators up to approximately 12.0 French may be passed through the working channel of the cystoscope. However, the telescope and inner scope elements must be removed, leaving the sheath, to accommodate the larger sizes.

FASCIAL DILATORS

1. Introduce the flexible tip end of the wire guide into an 18 gage needle previously positioned into a cavity under fluoroscopic control.
CAUTION: Once the wire guide is in position, the puncture needle is removed from the wire guide prior to introduction of dilators over the wire guide.
2. Progressing from the smallest to the largest appropriate size, pass the dilators over the wire guide while maintaining the wire guide's position.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd