- EN**
3 **Williams Cystoscopic Injection Needle**
Instructions for Use
- CS**
3 **Cystoskopická injekční jehla Williams**
Návod k použití
- DA**
4 **Williams cystoskopiske injektionsnål**
Brugsanvisning
- DE**
5 **Williams Zystoskop-Injektionskanüle**
Gebrauchsanweisung
- EL**
6 **Κυστεοσκοπική βελόνα έγχυσης Williams**
Οδηγίες χρήσης
- ES**
7 **Aguja de inyección cistoscópica Williams**
Instrucciones de uso
- FR**
8 **Aiguille pour injection cystoscopique de Williams**
Mode d'emploi
- HU**
9 **Williams cisztoszkópai injekciós tű**
Használati utasítás
- IT**
10 **Ago per iniezione cistoscopica Williams**
Istruzioni per l'uso
- NL**
11 **Williams cystoscopische injectienaald**
Gebruiksaanwijzing
- NO**
12 **Williams cystoskopisk injeksjonsnål**
Bruksanvisning
- PL**
13 **Cystoskopowa igła Williams do wstrzyknięć**
Instrukcja użycia
- PT**
14 **Agulha de injeção citoscópica Williams**
Instruções de utilização
- SV**
15 **Williams-cystoskopisk injektionsnål**
Bruksanvisning

WILLIAMS CYSTOSCOPIC INJECTION NEEDLE

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner). Please read all instructions before using this device.

DEVICE DESCRIPTION

The Williams Cystoscopic Injection Needle is a semi-rigid injection needle. French size, gage, length, and tip length will vary by configuration.

INTENDED USE

Used for cystoscopic injection into the urethra, bladder neck, and bladder wall.

CONTRAINDICATIONS

The device is contraindicated in the presence of conditions which create unacceptable risk to the patient during cystoscopy.

WARNINGS

None known

PRECAUTIONS

- Do not force the device during the procedure; stop and determine the cause of resistance before proceeding.
- This product is intended for use by physicians trained and experienced in cystoscopic techniques. Standard techniques should be employed.

POTENTIAL ADVERSE EVENTS

Complications associated with cystoscopy procedures may include, but are not limited to:

- Bleeding
- Urinary tract perforation
- Stricture
- Urinary retention

Additional complications associated with the urological injection needles may include, but are not limited to:

- Inadvertent puncture with tissue damage or bleeding
- Infection
- Inadvertent injection of the therapeutic agent

INSTRUCTIONS FOR USE

1. Securely attach a pre-filled syringe to the Luer lock fitting on device, then flush air from the needle. Check for patency and leaks.
2. With the cystoscope already placed in the urethra, introduce the needle into the working channel of the cystoscope.
3. Advance the needle in short increments until it is visualized exiting the cystoscope.

NOTE: Needle should be advanced no further than 5 cm from the end of the scope.

4. Cystoscopically visualize the desired injection site and maintain position of cystoscope.
5. Advance the needle into the intended injection site.
6. Perform injection(s) to desired location(s).
7. Upon completion of injections, carefully remove the needle from the cystoscope.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd