**COOK**

**MEDICAL**

- EN**  
3  
Wittich Nitinol Stone Basket Set  
Instructions for Use
- DA**  
3  
Wittich stenkurvsæt af nitinol  
Brugsanvisning
- DE**  
4  
Wittich Steinfangkörbchen-Set aus Nitinol  
Gebrauchsanweisung
- EL**  
5  
ΣΕΤ καλαθιού λίθων από νιτινόλη Wittich  
Οδηγίες χρήσης
- ES**  
6  
Equipo de cesta para cálculos de nitinol Wittich  
Instrucciones de uso
- FR**  
7  
Set de panier pour calculs Wittich en nitinol  
Mode d'emploi
- IT**  
8  
Set Wittich con cestello in nitinol per il  
recupero dei calcoli  
Istruzioni per l'uso
- NL**  
9  
Wittich set met nitinolnetje voor stenen  
Gebruiksaanwijzing
- PL**  
10  
Zestaw nitynolowego koszyka do kamieni  
Wittich  
Instrukcja użycia
- PT**  
11  
Conjunto de cesto Wittich em nitinol para  
remoção de cálculos  
Instruções de utilização
- SV**  
12  
Wittich Nitinol stenkorgsset  
Bruksanvisning

T \_ W N S B \_ R E V 5

## WITTICH NITINOL STONE BASKET SET

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).**

### DEVICE DESCRIPTION

The Wittich Nitinol Stone Basket Set consists of an introducer sheath with a radiopaque band, a basket catheter and a loading sleeve. The basket is made of nitinol with a six-wire bulb configuration.

### INTENDED USE

The Wittich Nitinol Stone Basket Set is intended for placement through a percutaneous tract for removal of stones (calculi) or stone fragments from anatomic structures such as the gallbladder, biliary tract, renal pelvis and ureter.

### CONTRAINDICATIONS

None known

### WARNINGS

None known

### PRECAUTIONS

- Imaging guidance is required when using this product.
- This product is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard techniques for placement of access sheaths, catheters, and wire guides should be employed.

### INSTRUCTIONS FOR USE

1. Prep and drape the patient according to hospital protocols.
2. Insert 0.035 inch or 0.038 inch wire guide through a pre-placed access catheter or sheath into the target anatomic structure. **NOTE:** Placement of a second "safety" wire through the access sheath should be considered.
3. Maintain wire guide position within the target structure, and remove the access catheter or sheath.
4. Insert the introducer sheath over the wire guide and across the percutaneous tract into anatomic structure. Position the radiopaque band of the sheath at or beyond the stone(s).
5. Remove the wire guide and dilator from the sheath, leaving the sheath in position.
6. Slide the loading sleeve down the basket catheter to cover the basket portion of the basket catheter.
7. Insert the loading sleeve and basket catheter into the hub of the introducer sheath.
8. Advance the basket catheter through the sheath and position the basket beyond the stone(s).
9. Rotate and withdraw the basket catheter at the level of the stone(s) to position/capture the stone(s) within the basket.
10. Once the stone(s) are captured, withdraw the basket and sheath from the tract.
11. If multiple passes are needed, repeat steps 3-10 until no additional stones can be captured.
12. At completion of the procedure, a temporary external drainage catheter may be positioned within the target structure if desired.

### HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

### REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.