Syfonix™ Suction Ureteral Access Sheath

FS

INSTRUCTIONS FOR USE

- PLEASE READ THESE INSTRUCTIONS CAREFULLY BEFORE USING THIS PRODUCT.
- KEEP THESE INSTRUCTIONS FOR REFERENCE.
- IN CASE OF ANY SERIOUS INCIDENT PLEASE REPORT IT TO THE MANUFACTURER AND YOUR COMPETENT AUTHORITY.
- THESE INSTRUCTIONS FOR USE ARE BASED ON EXPERIENCE FROM PHYSICIANS AND/OR THEIR PUBLISHED LITERATURE. IT IS INTENDED TO ASSIST IN USING THIS DEVICE. IT IS NOT A REFERENCE TO SURGICAL TECHNIQUES.

Date of issue: 2025-12-16

Version: A1-Cook

Contents / Contenu

1. DEVICE NAME / NOM DU DISPOSITIF	2/5
2. DESCRIPTION / DESCRIPTION	2/5
3. INTENDED USE/INDICATIONS / INTENDED USE/INDICATIONS	2/5
4. STORAGE / STOCKAGE	2/5
5. SPECIFICATION / SPÉCIFICATION	2/5
6. CONTRAINDICATIONS / CONTRE-INDICATIONS	3/6
7. DIRECTIONS FOR USE / MODE D' EMPLOI	3/6
8. WARNINGS/PRECAUTIONS / AVERTISSEMENTS/PRÉCAUTIONS	3/6
9. POTENTIAL COMPLICATIONS / COMPLICATIONS POTENTIELLES	4/7
10. EXPLANATION OF SYMBOLS / EXPLICATION DES SYMBOLES	4/7

1. DEVICE NAME

Syfonix™ Suction Ureteral Access Sheath

2. DESCRIPTION

The ureteral access sheath consists of a ureteral access sheath and a tapered dilator with locking hub. Hydrophilic coating of the ureteral access sheath minimizes shear force and enabled smoother passage of the sheath into the ureter. The sheath's locking mechanism on the dilator hub secures the dilator to the sheath for simultaneous advancement of the sheath and dilator.

Kink-resistant sheath – reinforced with a stainless steel coil, designed to provide stability and strength through tortuous anatomy. The ergonomic funnel acts as a handle during insertion. The large trough facilitates instrument introduction.

The device is delivered in a sterile state with EO sterilization. The shelf life is defined for 3 years after sterilization. A re-sterilization by users is not allowed.

3. INTENDED USE/INDICATIONS

The ureteral access sheath is used to establish a working channel during endoscopic urological procedures, facilitating the passage of endoscopes and other instruments into the urinary tract. The intended population of this device should be adult only.

This is a single-use device. A re-sterilization by users and re-use of the device are not allowed.

4. STORAGE

The device should be stored at a temperature of 0-40°C and be kept away from the sunlight, the appropriate relative humidity should be ≤80%. The shelf life of the device is 3 years.

5. SPECIFICATION

The ureteral access sheath is to be introduced in retrograde ureteroscopy approach. A guide wire with a diameter of .035 inch (0.89 mm) or .038 inch (0.97 mm) is needed.

Variants table

Description	Fr	Sheath Outer diameter-D1	Sheath Inner Diameter-D2	Sheath Length	Dilator Outer Diameter	Length of dilator extending out of the sheath
SYUAS-9-40	9F	3.66mm	3.0mm	40cm	3.0mm	25mm
SYUAS-9-50	9F	3.66mm	3.0mm	50cm	3.0mm	25mm
SYUAS-10-40	10F	4.0mm	3.33mm	40cm	3.33mm	25mm
SYUAS-10-50	10F	4.0mm	3.33mm	50cm	3.33mm	25mm
SYUAS-11-40	11F	4.33mm	3.66mm	40cm	3.66mm	25mm
SYUAS-11-50	11F	4.33mm	3.66mm	50cm	3.66mm	25mm
SYUAS-12-40	12F	4.66mm	4.0mm	40cm	4.0mm	25mm

Description	Fr	Sheath Outer diameter-D1	Sheath Inner Diameter-D2	Sheath Length	Dilator Outer Diameter	Length of dilator extending out of the sheath
SYUAS-12-50	12F	4.66mm	4.0mm	50cm	4.0mm	25mm

6. CONTRAINDICATIONS

Patients who cannot tolerate endoscopic urological surgery.

Individuals with lower urinary tract obstruction.

Individuals who are allergic to device materials.

Patients with long ureteral stenosis, severe occlusion, and severe fibrosis around the ureter.

7. DIRECTIONS FOR USE

NOTE: Prior to placement, activate the hydrophilic coating by immersing the outer sheath and dilator tip in sterile saline. This will allow the hydrophilic surface to absorb water and become lubricious, easing placement under standard conditions.

- 1) According to the use requirements, select the appropriate size of the ureteral access sheath.
- 2) Open the sterile packaging and take out the device under aseptic conditions. Taking care to prevent contamination of the device.
- 3) Place a guide wire into the ureter via urethra (retrograde ureteroscopy approach). Guide wire with a diameter of .035 inch (0.89 mm) or .038 inch (0.97 mm) can be used.
- 4) Grasp the sheath just below the instrument adapter and advance the dilator/sheath assembly over the guide wire and into the ureter.

NOTE: Be sure the dilator is securely locked onto the instrument adapter, ensuring the dilator/sheath assembly can be placed as a single unit, allowing one-hand placement.

- 5) Confirm the dilator/sheath assembly is properly placed via fluoroscopy.
- 6) While holding the sheath in position, unlock the fitting and remove the dilator.
- 7) Introduce the desired endoscope or instrument as needed. When used in conjunction with an endoscope, the deflecting tip of the sheath enters the renal pelvis with the endoscope to establish a channel for clearing stones. During the surgery, the drainage fluid can flow out through the oblique tube at the sheath socket and filter the stones through a negative pressure drainage bottle.

8. WARNINGS/PRECAUTIONS

- 1) Read the instructions for use carefully before use.
- 2) Ureteral access sheath is provided in sterile state and intended for use in a single procedure only. Do not re-use, reprocess or resterilize. An infection or transmission of diseases could occur, if the device were to be re-used.
- 3) Do not use after expired date.
- 4) Never use undue force for placement of this device.
- 5) Inspect the integrity of the packaging before use. Do not use if package is open or damaged.
- 6) Upon removal from the package, inspect the product to ensure no damage has occurred.
- 7) Endoscopic procedures should be performed only by physicians with adequate training and knowledge of these procedures. In addition, medical literature should be consulted regarding techniques, hazards, contraindications and complications prior to the performance of these procedures.

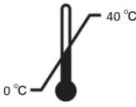
- 8) To prevent ureteral wall injury, surgeons should carry a low threshold for switching to a smaller sheath when resistance is felt or if placement time is long.
- 9) Do not force components during removal. Carefully remove the components if any resistance is encountered.
- 10) After use this device maybe a potential biohazard. Handle and dispose of in accordance with acceptable medical practices and with applicable local, state and federal laws and regulations.

9. POTENTIAL COMPLICATIONS

Complications of ureteral access sheath are documented. These complications may include:

- Ureteral trauma, e.g. ischemic injury
- Ureteral wall injury
- Urinary tract infection

10. EXPLANATION OF SYMBOLS

	Sterilized using ethylene oxide		Do not re-use		Do not re-sterilize
	Catalogue number		Batch code	Rx Only	Caution: Federal law restricts this device to sale by or on the order of a physician
	Use-by date		Do not use if package is damaged and consult instructions for use		Unique device identifier
	Medical device		Manufacturer		Caution
	Consult instructions for use or consult electronic instructions for use		Keep away from sunlight		Keep dry
	Temperature Limit 0-40 °C		Humidity limitation ≤ 80 %		Single sterile barrier system with protective packaging outside
	Single sterile barrier system with protective packaging inside				

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Distributed by:

Cook Medical, LLC
400 Daniels Way
Bloomington, IN 47404

Manufacturer Information for US

Zhejiang YiGao Medical Technology Co., Ltd.
No. 3, Gaoxin Jiu Road, Xiaoshan Economic and Technological Development Zone, Hangzhou 311200 Zhejiang, P.R. China

Manufacturer Information for Canada

Zhejiang YiGao Medical Technology Co., Ltd.
Room 749, 7th Floor, No. 6, Yinhu Innovation Center, No. 9 Fuxian Road, Yinhu Street, Fuyang District Hangzhou, Zhejiang ,311400 , CHINA

Manufacturer Information for Australia

Zhejiang YiGao Medical Technology Co., Ltd.
No. 3, Gaoxin Jiu Road, Xiaoshan Economic and Technological Development Zone, Hangzhou 311200 Zhejiang, P.R. China

Australian Sponsor

CLARO MEDTECH PTY LTD
Unit 605/12 Albert Street, Hawthorn East, VIC 3123, Australia
info@claromedtech.com.au