EN
2
CS
9
DA
16
DE
23
EL
30
ES
37
FR
44
HU
51
IT
58
NL
65
NO
72
PL
79
PT
86
SV
93
ZH
100

LMA Stone Breaker™
PNEUMATIC LITHOTRIPTOR

LMA Stone Breaker™
PNEUMATICKÝ LITOTRYPTOR

LMA Stone Breaker™
TRYKLUFTDREVET LITHOTRIPTOR

LMA Stone Breaker™
PNEUMATISCHER LITHOTRIPTOR

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LMA Stone Breaker™
气动碎石器

DEVICE DESCRIPTION 2

- Introduction 2
- Components and Accessories 2

WARNINGS AND CAUTIONS 3**INTENDED USE 3****INDICATION FOR USE 3****ADVERSE EFFECTS 3****PREPARATION FOR USE 3**

- Preparation for Cleaning, Disinfection and Sterilisation 3
- Cleaning 4
- Disinfection 4
- Sterilisation 4

PROCEDURE 4

- General 4
- Pre-use Preparation 4
- Intracorporeal Lithotripsy Procedure 4

MAINTENANCE 5**EFFECTS ON ENVIRONMENT 5****RETURN TO MANUFACTURER 5****MANUFACTURER'S WARRANTY 5****TROUBLESHOOTING 6****TECHNICAL DATA 7**

- LMA StoneBreaker™ and Probes 7
- LMA StoneBreaker™ CO₂ Cartridge 7

Copyright © The Laryngeal Mask Company Limited, 2009

All rights reserved. No part of this publication may be reproduced, stored in a retrieval system or transmitted in any form or by any means electrical, mechanical, photocopying, recording or otherwise, without the prior permission of the publisher. LMA, LMA StoneBreaker, The Laryngeal Mask Company Limited logo and its component parts are trademarks of The Laryngeal Mask Company Limited. The information given in this document is correct at the time of going to press. The manufacturer reserves the right to improve or modify the products without prior notification.

LMA StoneBreaker™ DEVICE DESCRIPTION

Introduction

The LMA StoneBreaker™ is a portable, non-electrical, compact intracorporeal lithotripter, intended for use to fragment stones in the urinary tract (i.e. kidney, ureter and bladder). It weighs around 500g with a probe and it requires no extraneous electrical or pneumatic connections. It is powered by a detachable cartridge of high pressure carbon dioxide gas, with a maximum pre-adjusted operating pressure of 31 bar. The mechanical shock generated by the device is transferred as kinetic energy through the length of the probe to the tip, which is in direct contact with the stone to be fragmented. The advantage of delivering high pressure at contact is to allow expeditious fragmentation, thereby reducing operative time. There is very little movement of the probe itself, allowing for safe stone fragmentation.

A LMA StoneBreaker™ carbon dioxide (CO₂) gas cartridge typically provides the necessary energy for one surgical procedure. Once the cartridge is perforated by the built-in screw system, the compressed gas passes through a pre-adjusted pressure regulator in the device and a mechanical shockwave required for fragmentation of urinary stone is generated when the trigger is depressed. Expended gas will leave the device through a specially designed exhaust port and the attached exhaust line. The trigger snaps back to its starting position when the gas is expended through the exhaust port, and the device is now ready for further use.

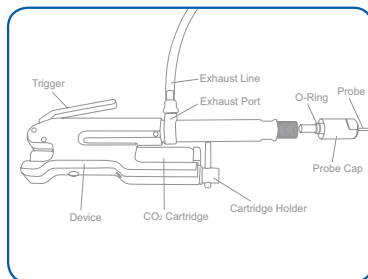
Figure 1

One full LMA StoneBreaker™ carbon dioxide gas cartridge should allow delivery of at least 80 shocks. The cartridge must be removed from the device after the completion of the surgical operation to allow complete release of gas from the cartridge.

The LMA StoneBreaker™ is recommended for use with endoscopes equipped with a straight working channel.

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a practitioner licensed by state law to use such device.

WARNING: Please read this instruction manual carefully before using the LMA StoneBreaker™.

Components and Accessories

The LMA StoneBreaker™ device is made up of the following main components. (See Figure 1)

- Trigger – Generates the mechanical shockwave required for the fragmentation of the calculus.
- Probe Cap – This holds the probe in place.
- Cartridge Holder – This is to lock and hold the carbon dioxide gas cartridge in place.
- Carbon Dioxide Gas Cartridge – This is the energy source. The LMA StoneBreaker™ carbon dioxide gas cartridges, which are sterilised by Gamma irradiation and are supplied separately from the device, are surface-sterile only, the gas within the cartridge is not sterile.

CAUTION: The gas cartridge must be removed from the device after the completion of each surgical procedure.

- Exhaust Port – This is for the exhaust of expended carbon dioxide gas from the device.
- Exhaust Line – This flexible tube is attached to the exhaust port to lead the expended gas away from the field of operation.
- Probe – The LMA StoneBreaker™ Probes are supplied in separate stand-alone packaging. The probes are single-use and are sterilised by Gamma irradiation.

Table 1 indicates the current LMA StoneBreaker™ Probes available for use with the device. Please refer to the product catalogue for the most up-to-date available product range.

Table 1: Specification of LMA StoneBreaker™ Probes

Description	Diameter	Total Length
Probe 1.0 mm	1.0 mm (3 Fr)	500 mm (19.7")
Probe 1.0 mm	1.0 mm (3 Fr)	605 mm (23.82")
Probe 1.6 mm	1.6 mm (4.8 Fr)	500 mm (19.7")
Probe 1.6 mm	1.6 mm (4.8 Fr)	605 mm (23.82")
Probe 2.0 mm	2.0 mm (6 Fr)	425 mm (16.73")

The following accessories are included with the LMA StoneBreaker™:

- Exhaust Cap – Closes the Exhaust Port to prevent moisture from entering the device during cleaning, disinfection and sterilisation.
- Sterilisation Cap – Replaces the carbon dioxide gas cartridge to prevent moisture from entering the device during cleaning, disinfection and sterilisation.

CAUTION: Use only LMA StoneBreaker™ CO₂ cartridges and probes; the use of the wrong gas cartridges and/or probes can affect the device performance and patient safety.

WARNINGS AND CAUTIONS

- The LMA StoneBreaker™ must be used only by specialists trained in endourology and should be used only for the purposes intended. In case of malfunction or failure of the device, it is recommended that the surgeon has another unit (or other intracorporeal lithotripsy device) on standby in order to complete the treatment scheduled at the time.
- It is recommended that urologists who have not used an intracorporeal lithotripter before, to execute a few manipulations in vitro with a stone model in order to learn how to manipulate the device.
- Ensure that the probe tip does not exceed the endoscope channel at the time of introducing the endoscope into the urinary tract.
- The LMA StoneBreaker™ is delivered non-sterile and it must be cleaned, disinfected and sterilised before initial use and before each subsequent use.
- For the LMA StoneBreaker™ Probe Single Use (sterile) : Do not use if the unit pouch has been previously opened or damaged.
- Do not use the single-use probe more than once. Re-usage of single-use probe increases the risk of patient injury, damage to device or endoscope or may lead to device malfunction.
- For the LMA StoneBreaker™ carbon dioxide gas cartridge (sterile), do not use if the unit pouch has been previously opened or damaged.
- Do not use the LMA StoneBreaker™ or any of the accessories if they are damaged in any way.
- Do not use the LMA StoneBreaker™ Probe if the O-Ring is missing or if it is damaged in any way.
- It is recommended that extra probes should be available in case of malfunction.
- Ensure the probe size chosen is compatible with the endoscope's working channel and appropriate for the clinical condition that is being treated.
- Do not autoclave the carbon dioxide gas cartridge with the LMA StoneBreaker™.
- Never attempt to refill the empty carbon dioxide gas cartridge.
- Never attempt to transfer gas from one carbon dioxide gas cartridge to another.
- Never attempt to disassemble the device.

INTENDED USE

The LMA StoneBreaker™ is intended for the fragmentation of urinary (i.e. kidney, ureter, and bladder) tract stones.

INDICATION FOR USE

The LMA StoneBreaker™ is indicated for use via a rigid or semi-rigid endoscope with a straight working channel, for the fragmentation of urinary (i.e. kidney, ureter and bladder) tract stones.

ADVERSE EFFECTS

There are no known recorded adverse effects with the use of the LMA StoneBreaker™. However, due to the movement of the endoscope and high force generated by the LMA StoneBreaker™ device, there is the potential of tissue injuries, should the probe tip be in direct contact with the mucosal membrane, causing perforation of tissue, pain, bleeding, renal damage and/or infection.

Review of published literature showed that adverse events that may potentially occur with the use of intracorporeal lithotripter include perforation of the bladder or ureter, bleeding in urinary tract or blood in the urine, pain, renal damage and infection.

Breakage of probe during use, though unlikely, may potentially occur. The practitioner shall then determine the most effective way of intervention, following all relevant procedures for such intervention.

PREPARATION FOR USE

Before each use, it is necessary to clean, disinfect and sterilise the LMA StoneBreaker™ device and exhaust line. Proper cleaning, disinfection and sterilisation are essential to ensure continued safe use of the device and accessories.

NOTICE: The LMA StoneBreaker™ Probe Single Use (sterile) and LMA StoneBreaker™ carbon dioxide gas cartridge (sterile) are individually packed, Gamma-sterilised and ready to use. The following procedure on cleaning, disinfection and sterilisation is not applicable to these sterile, "ready-to-use" products.

Preparation for Cleaning, Disinfection and Sterilisation

Ensure the sterilisation cap and exhaust cap are screwed tightly in place at the cartridge holder and exhaust port respectively. This is to prevent moisture from entering the device during cleaning, disinfection and sterilisation.

NOTICE: It is possible for moisture to adversely affect the effective performance of the device. If liquid has penetrated the device, return it to the manufacturer for servicing.

The hospital's own protocol for cleaning, disinfection and sterilisation of surgical instruments should be followed whenever possible. The following is the recommended procedure for cleaning, disinfecting and sterilising the LMA StoneBreaker™ device and exhaust line.

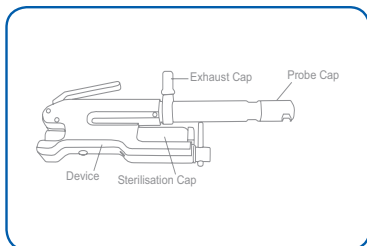
Figure 2

Cleaning

Ensure the probe is not attached to the device. If the probe is still present, unscrew the probe cap to remove the probe. Re-attach the probe cap back onto the device. Refer to Figure 2.

Thoroughly wash the LMA StoneBreaker™ and its accessories by gently cleaning with a soft bristle brush in warm water using mild detergents or enzymatic cleaning agents in accordance with the manufacturer's instructions until all visible foreign matter is removed.

Thoroughly rinse the device and accessories in warm running tap water to remove cleaning residue. Carefully inspect to ensure that all visible foreign matter has been removed. A specific cleaning agent found to be compatible with the LMA StoneBreaker™ and its accessories is Endozime® (Ruhof, Valley Stream, NY).

Disinfection

Submerge the LMA StoneBreaker™ device and its accessories into a disinfectant solution in accordance to the manufacturer's instructions.

CAUTION: Ensure the device is watertight by tightening the Sterilisation Cap and Exhaust Cap at their respective locations. One disinfecting solution that is found to be compatible with the LMA StoneBreaker™ device and accessories is Cidex® OPA (Johnson & Johnson, UK). Submerge the device and accessories into the Cidex® OPA at room temperature for at least 20 minutes. Thoroughly rinse the disinfected device with sterile water and dry it.

Sterilisation

Steam autoclaving is the only recommended method for sterilising the LMA StoneBreaker™ device and exhaust line. The device and its accessories should be placed in a suitable sterilisation bag prior to placing them in a steam autoclave. Steam autoclave the device and accessories by following the recommendations of the hospitals or the autoclave manufacturer. All steam autoclave cycles typically used for instruments and porous items are acceptable. One steam sterilisation cycle that is suitable for the LMA StoneBreaker™ device and exhaust line is to expose the device and accessories to steam at 134°C with a hold time of at least 10 minutes.

WARNING: Do not sterilise the CO₂ cartridge in an autoclave as the cartridge will explode when heated.

CAUTION: After autoclave sterilisation, do not utilise the LMA StoneBreaker™ in a warm condition, as this may cause damage to the components within. The following minimum cooling period is required: EITHER ambient air cooling period - 60 minutes minimum OR sterile water cooling period - 5 minutes minimum.

PROCEDURE

General

The LMA StoneBreaker™ requires effective endoscope manipulation in order to ensure proper direct contact of the probe with the calculus, with the probe tip protruding from the endoscope working channel by 10mm to 20mm maximum. The choice of the endoscope depends on the position of the stone (kidney, ureter or bladder). The LMA StoneBreaker™ is recommended for use with a rigid or semi-rigid endoscope equipped with a straight working channel, and to use the largest probe possible to allow transmission of maximum energy. Table 2 shows the recommended diameter for the endoscope working channel with the respective LMA StoneBreaker™ probes.

Table 2: Recommended Endoscope Working Channel Diameter

Probe Diameter	Diameter of Endoscope Working Channel
2.0 mm (6 Fr)	>2.33 mm (7 Fr)
1.6 mm (4.8 Fr)	>1.83 mm to 2 mm (5.5 to 6 Fr)
1.0 mm (3 Fr)	>1.17 mm to 1.8 mm (3.5 to 5.4 Fr)

NOTICE: It is recommended that for better and simplified alignment of the working channel and the LMA StoneBreaker™ Probe, endoscope adaptors that are available from the endoscope manufacturers should be used. Each carbon dioxide gas cartridge should provide sufficient energy for one typical procedure.

Pre-use Preparation

Prior to the use of the LMA StoneBreaker™ for stone fragmentation, it is recommended that the following checks are carried out:

- Check that the LMA StoneBreaker™ device and its accessories are not damaged in any way.
- For the LMA StoneBreaker™ Probe Single Use (sterile) and the LMA StoneBreaker™ carbon dioxide gas cartridge (sterile), check to ensure the unit pouch is not damaged or previously opened. Do not use if the unit pouch is damaged or previously opened.
- Remove both the Sterilisation Cap and Exhaust Cap from the LMA StoneBreaker™ device.
- Attach the Exhaust Line onto the Exhaust Port.
- Check and ensure the O-Ring is in the correct position on the probe.
- Remove the Probe Cap from the device and insert the probe through the opening in the Probe Cap. Replace the Probe Cap to the device and ensure that the probe does not move too freely.
- Insert a new LMA StoneBreaker™ carbon dioxide gas cartridge into the Cartridge Holder and tighten to hold the cartridge in place but NOT perforating it.

CAUTION: Do NOT perforate the cartridge at this point. Leaving a perforated carbon dioxide gas cartridge in the device for too long may adversely affect the performance of the device.

Intracorporeal Lithotripsy Procedure

Before the operation, tighten the cartridge holder until the cartridge is perforated. The trigger should now be under pressure. Turn the lever of the cartridge holder a quarter rounds more to secure the cartridge to the device.

CAUTION: Check that the device is working correctly by depressing the trigger two to three times and ensure the trigger returns to the original position every time the trigger is released. Position the Exhaust Line away from the head of the operator and ensure it is properly attached to the Exhaust Port.

CAUTION: Exhaust Line disconnection from the Exhaust Port during use may indicate malfunction of the Pressure Regulator.

Position the probe, through the endoscope working channel, to be in direct contact with the stone.

CAUTION: Ensure proper alignment of probe and endoscope. Any misalignment may damage both the probe and endoscope.

It is recommended that the practitioner ensures that the tip of the probe remains in sight throughout the procedure.

CAUTION: Ensure the probe is extended beyond the tip of the endoscope by 10mm to 20mm to avoid damage to the endoscope.

When ready, hold the device firmly in place and depress the trigger of the LMA StoneBreaker™, which will generate the mechanical shock required to fragment the targeted stone.

WARNING: Ensure minimal movement of the device and probe. Any significant movement of the device and probe during stone fragmentation may result in direct contact of the activated probe with tissue resulting in tissue damage and / or bleeding.

Release the trigger and allow it to return to its original position after each depression.

CAUTION: If the trigger cannot be depressed, it may indicate malfunction of the Pressure Regulator.

For subsequent shocks, reposition the device and hold it firmly before depressing the trigger as required.

NOTICE: One new full LMA StoneBreaker™ carbon dioxide cartridge should allow delivery of at least 80 shocks.

Once the procedure is complete, remove the carbon dioxide cartridge from the Cartridge Holder by loosening the lever slowly. This will allow any remaining gas in the cartridge to escape, thereby emptying the cartridge.

CAUTION: Do not sterilise the spent cartridge with the device.

MAINTENANCE

Though the LMA StoneBreaker™ is designed and manufactured to require minimum maintenance, proper care and maintenance are required to ensure safe and effective use of the device. The manufacturer recommends that the device and its accessories be carefully checked upon receipt and prior to each use to prevent injury to the patient or user. Follow the recommendations listed in this Instruction manual for proper routine maintenance and care of the device.

Defective probes (such as the presence of kink, bent or dent marks along the length of the probe) should be discarded. Other components that may be damaged and lose their efficiency in maintaining water-tightness of the device are the Sterilisation Cap and Exhaust Cap. Check the integrity of these components before using them on the device.

EFFECTS ON ENVIRONMENT

This device does not contain any harmful components; follow local statutory requirements / instructions for proper disposal of the non-functioning devices, probes and other accessories. Never dump the empty carbon dioxide cartridges at sea. The cartridge is made of zinc-plated and chrome stabilised steel, and should be properly disposed in accordance with local statutory requirements and instructions.

CAUTION: Never dispose an unused carbon dioxide cartridge. Ensure that the cartridge is empty before disposal.

RETURN TO MANUFACTURER

Should there be a need to return the LMA StoneBreaker™ to the manufacturer for investigation or repair, please ensure the product is properly cleaned, disinfected and sterilised in accordance to the recommendations listed in this instruction manual before shipment.

Please return device in its sterilisation bag (as evidence that the device is sterile), whenever possible. The manufacturer may refuse receipt of device and / or accessories if it is found that the products are not properly disinfected or sterilised or there is no evidence that it has been carried out.

MANUFACTURER'S WARRANTY

The manufacturer grants a one (1) year warranty from the date of invoice on the LMA StoneBreaker™. The manufacturer warrants the device against faulty materials and manufacturing defects, provided that the device is used in accordance with the instructions set forth in the instruction manual. If the device exhibits any such defects or damage, a new unit will be supplied. If covered by the warranty, this will be done free of charge; otherwise the customer will be invoiced for the cost incurred. This warranty is void if damage is caused by the improper use or treatment of the device, or damages due to external causes including accident, drops, alteration, or improper installation. Repair may only be carried out by the manufacturer or any of its authorised service centres. The warranty covering the reliability and proper functioning of the device is void if the user or any other unauthorised person tries to perform repairs. Single-use products are warranted against faulty materials or manufacturing defects at time of delivery to customer. Warranty is applicable only if products are purchased from an authorised distributor. The manufacturer disclaims all other warranties, whether express or implied, including, without limitation, the warranties of merchantability or fitness for a particular purpose.

TROUBLESHOOTING

Description	Possible Causes	Action
No shock generated	CO ₂ cartridge not perforated	Tighten the cartridge holder
	CO ₂ cartridge is empty	Replace cartridge
	Device malfunction	Return to manufacturer
	Probe Cap not properly in place	Tighten the probe cap
	Trigger not fully depressed	Depress trigger fully
Shock generated is weak or without force	CO ₂ cartridge is nearly empty	Replace cartridge
	CO ₂ cartridge not perforated properly	Tighten the cartridge holder fully
	Device malfunction	Return to manufacturer
Gas leakage	Cartridge not properly secured in the cartridge holder	Tighten the cartridge holder fully
	Device malfunction	Return to manufacturer
Shock generated has a very long excursion	Probe O-Ring is missing or damaged	Replace probe
No shock generated at the tip of the probe, when tested against a sterile gloved finger	Probe O-Ring is missing or damaged	Replace probe
Bad quality of the shock at the tip of the probe	Probe O-Ring is missing or damaged	Replace probe
Trigger activation too stiff or difficult to depress	Pressure regulator may be defective	Return to manufacturer
Trigger returns to original position too slowly	Device malfunction	Return to manufacturer
	CO ₂ cartridge is empty or nearly empty	Replace with a new cartridge
Exhaust line constantly gets disconnected from exhaust port when trigger is depressed	Pressure regulator may be defective	Return to manufacturer
	Exhaust line not properly attached to exhaust port	Tighten exhaust line to exhaust port
Black particles found inside the exhaust line after use	Foreign particles present within device, but the device is functioning well	Change the exhaust line

TECHNICAL DATA

LMA StoneBreaker™ and Probes

Operating mode	Intermittent, manually controlled by user			
Dimensions	Device	246 mm x 80 mm		
	Probe	Description	Diameter	Length
		Probe 1.0 mm	1.0 mm (3 Fr)	500 mm (19.7")
		Probe 1.0 mm	1.0 mm (3 Fr)	605 mm (23.82")
		Probe 1.6 mm	1.6 mm (4.8 Fr)	500 mm (19.7")
		Probe 1.6 mm	1.6 mm (4.8 Fr)	605 mm (23.82")
Probe 2.0 mm	2.0 mm (6 Fr)	425 mm (16.73")		
Weight	500g			
Conditions for use	Temperature range	+10°C to + 40°C		
	Relative humidity range	30% to 70% RH		
Transportation conditions	Temperature range	-10°C to + 45°C		
	Relative humidity range	10% to 95% RH		

LMA StoneBreaker™ CO₂ Cartridge

Content	Pressurised carbon dioxide gas (99.5% Purity)	
	UN code	1013
Dimension	Ø19 x 83 mm	
Weight	11.5 g	
Conditions for Use	Not higher than + 45°C	
Transportation conditions	UN Code	UN 2037
	UN Class	2.2
	Classification code	5A
	According to ADR chapter 3.2 schedule A column 6, the special provisions 191 and 303 apply: (transported product is not subject to the ADR, IMDG or RID regulations)	
Storage conditions	Temperature range	-30°C to + 45°C
	Store in a well-ventilated area and in a location free from fire risk and away from sources of heat and ignition	

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LMA StoneBreaker™ a sondy 14

Patrona CO₂ LMA StoneBreaker™ 14**Copyright © The Laryngeal Mask Company Limited, 2009**

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LMA StoneBreaker™-CO₂-magasin 21**Copyright © The Laryngeal Mask Company Limited, 2009**
LMA StoneBreaker™-CO₂-Kartusche 28**Copyright © The Laryngeal Mask Company Limited, 2009**
Cartucho de CO₂ LMA StoneBreaker™ 42**Copyright © The Laryngeal Mask Company Limited, 2009**
Cartouche de CO₂ LMA StoneBreaker™ 49**Copyright © The Laryngeal Mask Company Limited, 2009**
LMA StoneBreaker™ CO₂ patron 56**Copyright © The Laryngeal Mask Company Limited, 2009**
Cartucce di CO₂ LMA StoneBreaker™ 63**Copyright © The Laryngeal Mask Company Limited, 2009**